THUASNE

VENOFLEX	INCOGNITO ABSOLU
VENOFLEX	KOKOON
VENOFLEX	KOKOON ABSOLU
VENOFLEX	SIMPLY COTON FIN
VENOFLEX	SECRET
VENOFLEX	SECRET OPAQUE
VENOFLEX	CLINIC
VENOFLEX	SOFT & CARE
VENOFLEX	FAST LIN
VENOFLEX	FAST COTON
VENOFLEX	FAST LAINE
VENOFLEX	FAST AIR
VENOFLEX	CITY CONFORT
VENOFLEX	CITY CONFORT COTON
VENOFLEX	CITY CONFORT FIL D'ECOSSE
VENOFLEX	ÉLÉGANCE

fr	Chaussettes, bas-cuisse ou collant de compression médicale pour femme et homme	5
en	Medical compression socks, stockings or tights for women and men	6
de	Medizinische Kompressions-Kniestrümpfe, -Schenkelstrümpfe oder Strumpfhosen für Damen und Herren	7
nl	Medische compressiekousen, dijkousen of panty's voor dames en heren	8
it	Gambaletti, calze o collant per terapia compressiva da uomo e donna	9
es	Calcetines, medias o pantis de compresión para hombre y mujer	10
pt	Meias, raiz da coxa ou collants de compressão médica para senhora e homem	10
da	Medicinske kompressionsokker, -strømper eller -strømpbukser til damer og herre	11
fi	Lääkinnälliset kompressiosukat, -reisusukat tai -sukkahousut naisille ja miehille	12
sv	Medicinska kompressionsstrumpor, -lårstrumpor eller -strumpbyxor för dam och herr	13
el	Κάλτσες κάτω γόνατος, κάλτσες ριζομηρίου ή καλσόν ιατρικής διαβαθμισμένης συμπίεσης για άνδρες και γυναίκες	14
cs	Lýtkové punčochy, stehenní punčochy nebo zdravotní kompresní punčochové kalhoty pro ženy a muž	15
pl	Podkolanówki, pończochy lub rajstopy medyczne o stopniowanym ucisku dla kobiet i mężczyzn	16
lv	Medicīniskās kompresijas zeķes, garās zeķes vai zeķubikses sievietēm un vīriešiem	17
lt	Medicininės kompresinės vyru ir moterų puskojinės, kojinės iki klubo ar pėdkelnės	18
et	Raviotstarbeliselt survestavad sokid, sukad või retuusid naiste ja meeste jaoks	18
sl	Medicinske kompresijske nogavice, nadkolenke ali hlačne nogavice za ženske in moške	19
sk	Liečebné kompresívne podkolenky, stehenné pančuchy alebo pančuchové nohavice pre ženy a mužo	20
hu	Orvosi kompressziós zokni, harisnya vagy harisnyanadrág nők és férfiak számára	21
bg	Медицински компресивни чорапи, дълги чорапи или чорапогащници за жени и мъже	22
ro	Șosete, dresuri pentru coapsă sau colanți medicinali compresivi pentru femei și bărbați	23
ru	Медицинские компрессионные гольфы, чулки или колготки для женщин и мужчин	24
hr	Kompresijske medicinske dokoljenke, samostojeće natkoljenke ili hulahopke za žene i muškarc	25
zh	男女医用袜子、长筒袜或连裤袜	26
ko	성인남녀를 위한 무릎, 허벅지 또는 췌티형의 의료용 압박 스타킹	26
ar	جوارب، جوارب فخذية أو جوارب طويلة طبية ضاغطة للنساء والرجال	28

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en

MEDICAL COMPRESSION

SOCKS, STOCKINGS OR TIGHTS FOR WOMEN AND MEN

Description/Destination

VENOFLEX devices are intended for use on the lower limbs, by adults with the indications presented hereafter and whose measurements correspond to the size chart.

Composition

polyamide - élastane
 Components specific to certain product ranges (see label): viscose - linen - wool - cotton - silicone
 * Silicone for the stocking format

Properties/Mode of action

VENOFLEX devices provide compression by applying pressure to the limb, thereby promoting venous return.

Indications

- 10-15 mmHg/Class 1: restless legs, heavy legs, evening oedema, reticular varicose veins, telangiectasia, prevention, during prolonged standing, long journeys (DVT prevention), pregnancy and post-partum (prevention and treatment of minor chronic venous insufficiency and DVT prevention), orthostatic hypotension.
- May be used in superposition/combination with other compression devices if higher pressure level is needed.
- 15-20 mmHg/Class 2: heavy legs, restless legs, evening oedema, telangiectasia or reticular veins (< 3 mm), following sclerotherapy for telangiectasia or reticular veins (< 3 mm), following sclerotherapy or surgery for varicose veins (> 3 mm), varicose veins (> 3 mm), prevention of venous thrombosis in pregnant women or post-partum, prevention of venous thrombosis during long journeys, treatment of lymphoedema (maintenance phase).
- 20-36 mmHg/Class 3: varicose veins (> 3 mm), following sclerotherapy or surgery for varicose veins (> 3 mm), chronic venous oedema, trophic disorders (pigmentation, eczema), chronic trophic disorders (lipodermatosclerosis or atrophie blanche), healed venous ulcer, chronic venous disease associated with pregnancy or the post-partum period, prevention and/or treatment of venous thrombosis (lower limb) and post-thrombotic syndrome, lower limb lymphoedema (maintenance phase).

Contraindications

Known allergies to any of the components.
 Advanced diabetic microangiopathy (if pressure > 30 mmHg).
 Heart failure.
 Weeping or severe dermatological conditions of the leg.

Peripheral artery disease with an ABI < 0.6.
 Plegmiasa cerulea dolens (painful blue oedema with arterial compression).
 Septic thrombosis.

Precautions

Follow the advice of the healthcare professional who prescribed or supplied the product.

Choose the appropriate size to fit the patient, referring to the size chart.

In the event of discomfort, significant restriction, pain, change in limb volume or unusual sensations, remove the product and seek the advice of a health professional.

Do not apply the product directly to an open wound.

Before using the stockings, make sure any infected or open skin lesions are covered by a suitable dressing.

Check that the anti-slip system can be applied to healthy, unbroken skin.

It is recommended that the product be washed before it is used for the first time.

Verify the product's integrity before every use. For hygiene and performance reasons, do not re-use the product for another patient.

Venoflex Clinic: these products do not contain any metal wires liable to interfere with a magnetic knife. In the event of use of an electrostatic knife, it is recommended that a sock be worn rather than a stocking in order to keep the device away from the knife. The sock must be far enough away to ensure it does not come into contact with the plate or electrodes of the electrostatic knife.

Undesirable side-effect
 This device can cause skin reactions (redness, itching, blurs, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Instructions for putting on: handle the product with care when putting on (pay attention to long nails and jewelry).

How to put on VENOFLEX socks

1. Place your hand inside the sock and pinch the heel
 2. Slip your foot into the sock until the heel is correctly positioned
 3. Pull up to below the knee
 4. Rub your leg to spread the sock evenly
- How to put on VENOFLEX FAST socks*
1. Turn back the top of the sock to the heel and heel
 2. Slip your foot into the sock until the heel and heel are correctly positioned
 3. Unroll the sock from the ankle to below the knee

How to put on VENOFLEX SOFT&CARE socks

1. Pull over the heel
 2. Unroll the sock from the ankle to below the knee without pulling too much
 3. Rub your leg to spread the sock evenly
- How to put on VENOFLEX stockings or tight*
1. Place your hand inside the stocking and pinch the heel
 2. Slip your foot into the stocking or tight until the heel is correctly positioned
 3. Carefully pull up to the top of the thigh (do not pull too hard on the anti-slip band), or to the waist for tights
 4. Rub your leg to spread the stocking or tights evenly

Care

Machine-washable without fabric softener. Dry in the shade from direct sunlight or direct heat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations. Keep this instruction leaflet.

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©Thuasne - 2028201 (2021-06)